


Duplex Horizontal Tank Mount

1 - 5 HP, Desiccant

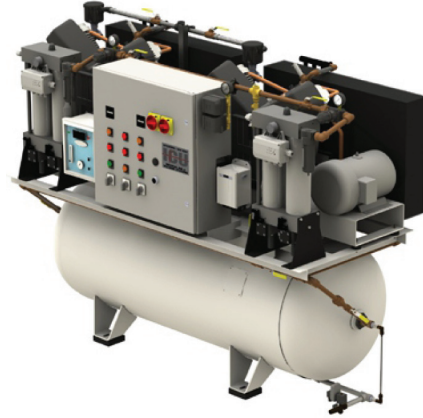

Description

NFPA 99 compliant duplex oil-less reciprocating medical air package. One piece tank mount system is designed (1 thru 5 hp) for single point connections. All systems are thoroughly tested prior to shipping. This system ships with inlet and discharge threaded flex lines vibration isolation pads and O&M manuals on CD and a hard copy.

Package Details

Tank mounted one piece medical air system including: Two each reciprocating air cooled oil-less air compressors with air cooled aftercoolers, automatic separator drains, and high temperature discharge switch. Air inlet filters with connections for remote intake. Two each TEFC electric motors, ASME tank with interior corrosion resistant lining, 3 valve bypass, sight gauge, electric automatic tank drain with manual drain valve and pressure safety valve. Dual desiccant air dryers with isolation valves, pre-filters with differential indicators and auto drains, 2-stage final filters with differential indicators. Pressure regulators with oxygen cleaned gauges, CO and Dew point monitors with audible/visual alarm and contacts for remote alarm and duplex NEMA 12 control panel. The package is pre-wired and pre-plumbed for single point connections.

Reciprocating Compressor Pump

The medical air compressors are a reciprocating, oil-less, air-cooled, 2-stage design (5 HP and above). The compressor has two way cooling with a dedicated cooling fan and fan flywheel. Permanently sealed lower end bearings, PTFE piston rings and robust inter-stage cooling ensures a long lived reliable pump.

Electric Motor/Drive

Each compressor is belt driven by a dedicated Premium Efficient 1,750 RPM totally enclosed, fan-cooled (TEFC) electric motor with a rigid base. Motor is 208-230/460 three phase with a service factor of 1.15.

Vibration Isolation

Each pump and motor assembly is mounted to the base frame/tank with seismically restrained vibration isolators.

Air Receiver

ASME 200 psi air receiver tank. The tank includes an interior corrosion resistant lining, 3 valve bypass, sight gauge, electric/automatic solenoid tank drain with manual drain valve and pressure safety valve.

Desiccant Air Dryers with Filters & Regulators

Duplex desiccant dryers with demand control purge saver. Each desiccant dryer is sized to meet the rated system flow capacity while maintaining pressure dew point in accordance with NFPA 99. Each dryer assembly is isolated with full port ball valves for easy isolation and maintenance. Each dryer includes: 0.01 micron pre-filters with differential indicators and auto drains, 2-stage final filters with a first stage rating of 3.0 micron, a secondary stage rating of 0.03 micron and a differential indicator. The dryers also include pressure regulators with oxygen clean gauges.

Dew Point & CO Monitors

The air system includes digital dew point and carbon monoxide monitors. Each monitor shall include audible and visual alarms, adjustable alarm set points and dry contacts for remote alarms.


Standard Control Panel

The system includes a duplex control panel in a NEMA 12 enclosure. The control panel is UL 508 listed and is designed to operate an NFPA 99 compliant medical air system. The panel includes: individual through-the-door disconnects for each motor, motor circuit breakers, magnetic starters, adjustable motor overload protection, dual control voltage transformers, alternating circuit, hand-off-auto switches, auto 1-2 pump selector switch, power on light, high temperature lights, transformer failure light, compressor run lights, pressure switches and alarm contact connection points. The control panel will start up immediately after power failure and no boot-up time is required.

Option

- Touch-Screen Interface Control System (HMI/PLC) with built-in ethernet, system monitoring, maintenance alerts and multiple protocols for facility automation systems.

Oil-less Reciprocating Compressor Package - Duplex Tank Mount 1 - 5 HP - Desiccant

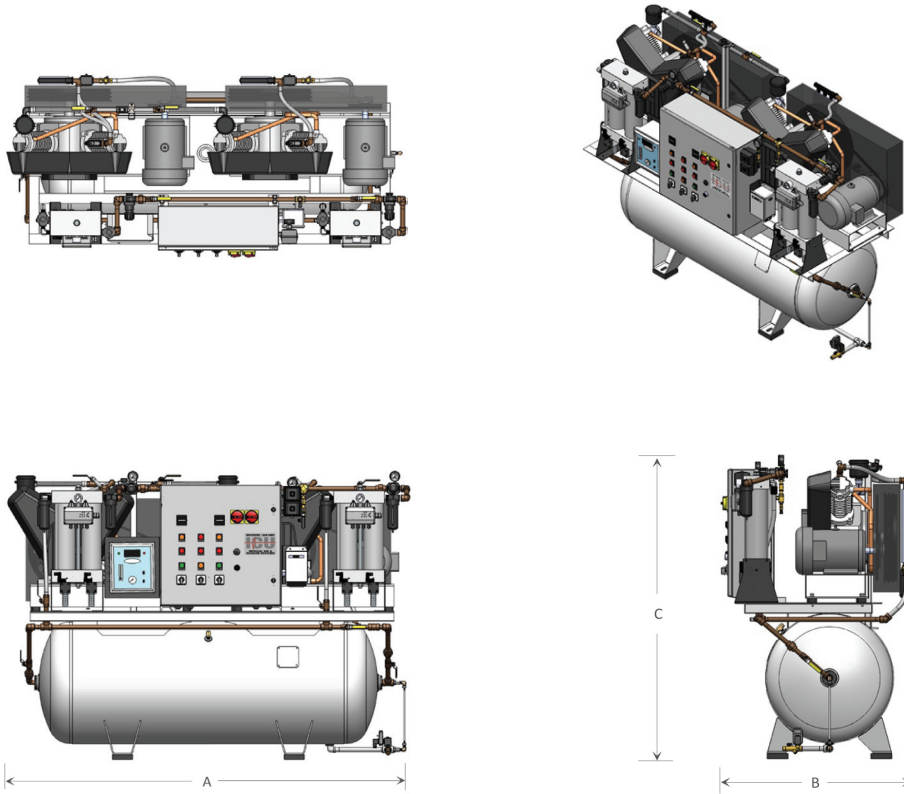

There are recommended minimum clearances for equipment maintenance and ventilation. Please contact us toll free at 800.333.0811

Shown: Duplex 5 HP, 120 gallon, horizontal tank mount, standard reciprocating piston configuration.

Model Number	Tank Gals	Horsepower		SCFM @ 50 ^(1,2)		dB(A) ⁽³⁾ Level	System FLA			BTU HR	Dimensions			NPT In	NPT Out	Weight (LBS)
		each	pkg	each	pkg		208V	230V	460V		A	B	C			
A71M-DTA-4HE-D	120	1.0	2.0	3.40	3.40	69	3.6	3.8	1.9	2,545	79.5	37.0	57.0	0.75	0.5	750
A71M-DTC-4HE-D	120	2.0	4.0	8.30	8.30	69	6.0	5.8	2.9	5,090	79.5	37.0	57.0	0.75	0.5	760
A71M-DTD-4HE-D	120	3.0	6.0	11.30	11.30	71	8.4	7.8	3.9	7,635	79.5	37.0	57.0	0.75	0.5	875
A71M-DTE-4HE-D	120	5.0	10.0	18.60	18.60	75	13.7	12.6	6.3	12,725	79.5	37.0	57.0	1.00	0.5	1,225

1. Compressor data is shown with one pump in reserve. 2. Compressor capacities are shown in SCFM (Standard Cubic Feet per Minute). 3. dB(A) levels are shown as only one pump running. 4. Data is shown with equipment running at normal operating conditions with a 105° maximum ambient temperature.

